Laerdal Airway Management Trainer

ENGLISH	Directions for Use
DEUTSCH	Gebrauchsanweisung
FRANÇAIS	Mode d'emploi
SVENSKA	Bruksanvisning
SUOMI	Käyttöohje
ESPAÑOL	Instrucciones de utilización
JAPANESE	取扱説明書

www.laerdal.com

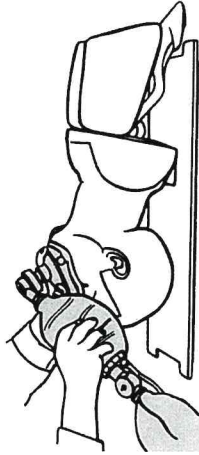
Laerdal
helping save lives

Laerdal
helping save lives

Ventilation

The Laerdal Airway Management Trainer allows use of all normal maneuvers to maintain an open airway; head tilt, chin lift, neck lift, jaw thrust. Ventilation can be practiced with or without equipment; mouth-to-mouth, mouth-to-nose, mouth-to-mask, bag-valve-mask.

The trainer's exterior and interior can be effectively cleaned after exhaled air resuscitation. (See under "Sanitation".)

Ventilation with excessive volume, or attempted against an obstructed airway, will cause inflation of the stomach. Make sure the elastic band is in place on the stomach retention valve.

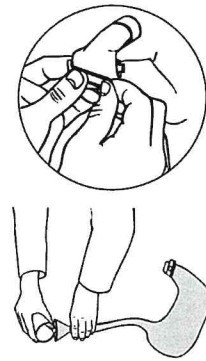
Note: Manikin exhales through mouth and nose.

Suction

The Laerdal Airway Management Trainer can be used for training in clearing the obstructed airway by suctioning liquid matter from:

- oral cavity
 - oro- or nasopharynx
 - oro- or nasotrachea, via endotracheal tube
 - bronchial, using the optional bronchial tree
- Gastric drainage may also be practiced.

Prepare simulated vomit. See mixing instructions on bottle. Remove the stomach retention valve. Fill stomach with approx. 4 cups of simulated vomit using a funnel inserted into the rigid connector.

Remount retention valve and connect stomach to esophagus. To induce vomiting press on the full stomach.

For a list of spare parts please visit www.laerdal.com.

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

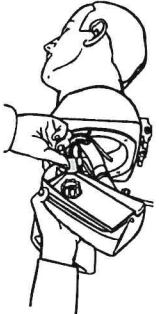
Stomach contents can be effectively removed from the interior of the trainer by flushing with clean water immediately after training session. Even stomach contents that have unintentionally been allowed to dry inside the trainer, will redissolve in water. To reduce dissolution time circulate water through the head using the cleaning pump assembly. See "Sanitation after training" point (B) below.

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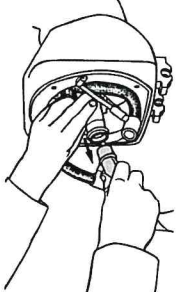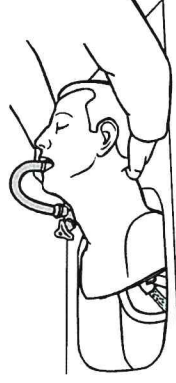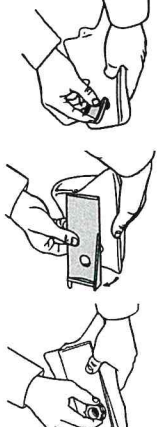
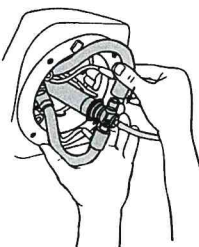
Lungs and stomach

Disassemble as described in previous section. Pour liquid into stomach and each lung.

Shake well. Flush with clean water. Allow all parts to dry before reassembly on trainer.

Preventive care

Avoid contact between manikin's skin and materials such as painted or lacquered surfaces, newsprint, ballpoint pens and lipstick. Advise trainees to remove lipstick and wash hands.

To wash the manikin's skin use warm water and a mild detergent.

Excessive force has no place during endotracheal intubation. Strong force can cause severe trauma to the patient and may also damage the plastic "tissue" of the manikin.

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